

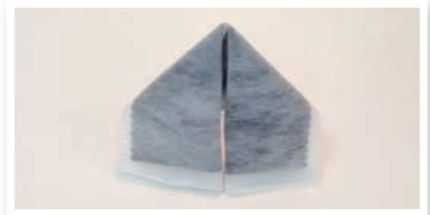
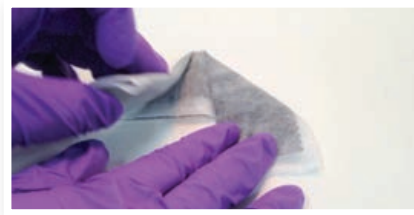
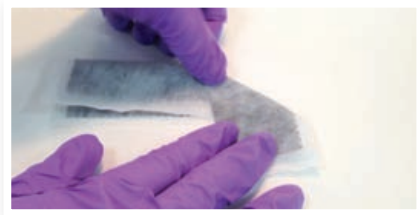


USER GUIDE FOR PEG SITES

ACTISORB[®] SILVER 220

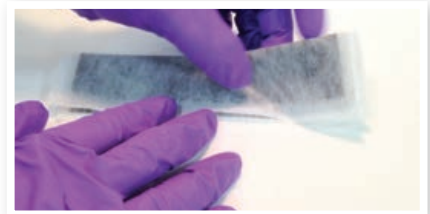

ACTISORB[®] Silver 220 reduces bacterial colonization in the dressing, the bacterial barrier properties of the dressing may prevent infection and overgranulation¹.

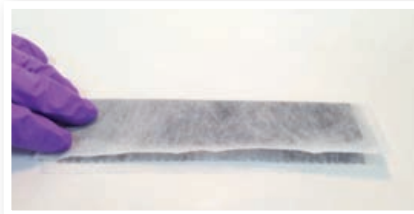

To learn more about the benefits of ACTISORB[®] Silver 220, please contact your local Systagenix representative at **1-866-978-3296** or visit us at www.systagenix.net

ACTISORB[®] SILVER 220

Item Code	Size	Eaches carton/box	HCPCS code
650220	2 1/2" x 3 3/4"	10 ea/ct – 5 ct/bx	A6206
105220	4 1/8" x 4 1/8"	10 ea/ct – 5 ct/bx	A6207
190220	4 1/8" x 7 1/2"	10 ea/ct – 5 ct/bx	A6207



www.systagenix.net

*Scan the QR code with your iPhone/Smart Phone to view the video: "How to use ACTISORB[®] folding technique" Alternatively, visit our website www.systagenix.net/AB-ACTISORB-Silver-220-Fold

The devices shown in the pictures are skin level and catheter.

References
1. Leak, 2002. PEG sites infections: a novel use for Actisorb Silver 220. British Journal of Community Nursing, Vol 7, No 6

© Systagenix Wound Management Limited 2014. Brands marked with ® or ™ are trademarks of Systagenix. All other products referenced herein are acknowledged to be trademarks of their respective owners.