

### TIELLE™ Border Dressings

- TIELLE™ Silicone Border Hydropolymer Dressing with Silicone with LIQUALOCK™ Technology
- TIELLE ESSENTIAL™ Silicone Border Silicone Adhesive Foam Dressing
- TIELLE ESSENTIAL™ Border Adhesive Foam Dressing

#### Prepare your wound *(follow your doctor's instructions)*

- Cleanse the wound
- Moisten the wound
- Other: \_\_\_\_\_  
\_\_\_\_\_

#### Apply your dressing *(see reverse side)*

1. Partially peel back the release liners on the dressing
2. Center the dressing over the wound, apply directly to wound while peeling away the release liner
3. Gently press and hold the dressing on the surrounding skin to secure


#### Change your dressing *(follow your doctor's instructions)*

- Remove dressing by peeling off from a corner using gentle pressure
- Change daily
- Change every other day
- Other: \_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

## TIELLE™ Border Dressings

### Apply your dressing

1. Partially peel back the release liners on the dressing

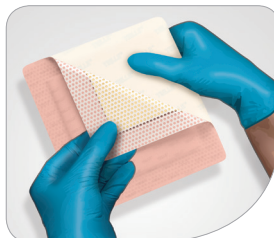

2. Center the dressing over the wound, apply directly to wound while peeling away the release liner



3. Gently press and hold the dressing on the surrounding skin to secure



### Remove your dressing

- Remove dressing by peeling off from a corner using gentle pressure



For more information, please contact Acclity at **800-275-4524** or visit **acclity.com**

**NOTE:** Specific indications, contraindications, warnings, precautions and safety information may exist for Systagenix products. Prior to use of a Systagenix product, it is important for the provider to consult the treating physician and read and understand all Instructions for Use, including Safety Information and Dressing Application Instructions. Rx only.