

TIELLE™ Hydropolymer Adhesive Dressings

- TIELLE™ Lite Hydropolymer Adhesive Dressing with LIQUALOCK™ Technology
- TIELLE™ Hydropolymer Adhesive Dressing with LIQUALOCK™ Technology
- TIELLE™ Plus Hydropolymer Adhesive Dressing with LIQUALOCK™ Technology

Prepare your wound *(follow your doctor's instructions)*

- Cleanse the wound
- Moisten the wound
- Other: _____

Apply your dressing *(see reverse side)*

1. Partially peel back side backing papers
2. Position absorbent island centrally over wound site and smooth in place
3. One at a time, peel away side backing papers while smoothing adhesive border onto intact skin

Change your dressing *(follow your doctor's instructions)*

- **IMPORTANT:** Adhesive border may be pre-moistened with water or saline prior to removal; this allows for quick/easy release

- Change dressing when wound fluid is present at the edges of the foam pad
- Change daily
- Change every other day
- Other: _____
- _____
- _____
- _____

TIELLE™ Hydropolymer Adhesive Dressings

Apply your dressing

1. Partially peel back side backing papers



2. Position absorbent island centrally over wound site and smooth in place



3. One at a time, peel away side packing papers while smoothing adhesive border on intact skin



Remove your dressing

- **IMPORTANT:**
Adhesive border may be pre-moistened with water or saline prior to removal; this allows for quick/easy release
- Change dressing when wound fluid is present at the edges of the foam pad



For more information, please contact Acelyty at **800-275-4524** or visit **acelyty.com**

NOTE: Specific indications, contraindications, warnings, precautions and safety information may exist for Systagenix products. Prior to use of a Systagenix product, it is important for the provider to consult the treating physician and read and understand all Instructions for Use, including Safety Information and Dressing Application Instructions. Rx only.



Copyright 2016 Systagenix Wound Management, Limited. All rights reserved. All trademarks designated herein are proprietary to Systagenix Wound Management, Limited, its affiliates and/or licensors. PRB C5542 • LIT 29-A-307 (Rev. 12/16)

