

# KerraFoam®

## Gentle Border

# All foams soak it up, KerraFoam® *locks it up*

### LOCKS IN MOISTURE AND HELPS TO REDUCE MACERATION

Older generation foam dressings claim superior fluid handling by virtue of their MVTR rates not absorption rates. However, MVTR is fine under laboratory conditions but in a clinical situation, with changes in humidity under bandages and temperature changes means that MVTR can reduce, substantially increasing risk of maceration!

### EXCELLENT RETENTION CAPABILITIES

KerraFoam Gentle Border also has excellent fluid retention capabilities in comparison to other foam dressings - **retaining more than double that of the two leading foam brands.**<sup>2</sup> Instantly locking harmful excess exudate away from the skin helps to **prevent maceration and wound edge breakdown.**

### HIGHLY COMFORTABLE AND CONFORMABLE

Soft silicone adhesive helps to prevent skin stripping.<sup>3</sup>  
Adheres well to awkward areas without limiting movement.



# Designed for protection and exuding wounds, secured with a gentle silicone contact layer.

## 1 SLIDES AND PROTECTS

The smooth outer film layer helps to reduce shear and friction<sup>5</sup>, create a water-resistant<sup>6</sup> bacterial barrier<sup>7</sup>, and maintain an ideal MVTR.<sup>4</sup>

## 2 GLIDES

A soft inner pouch allows the layers to move freely, absorbing shear and friction forces that can damage peri-wound skin.<sup>5</sup>

## 3 ABSORBS AND LOCKS


Absorbent core and foam pad combine to lock in exudate<sup>1</sup>, bacteria<sup>2</sup> and MMPs<sup>3</sup> - even under compression.<sup>1</sup>

## 4 WICKS

Our unique horizontal wicking layer efficiently increases absorption capacity and reduces pressure points by distributing exudate throughout the dressing.<sup>8</sup>

## 5 STICKS

A soft silicone layer ensures gentle adhesion, excellent absorption rate and non traumatic removal.<sup>9</sup>

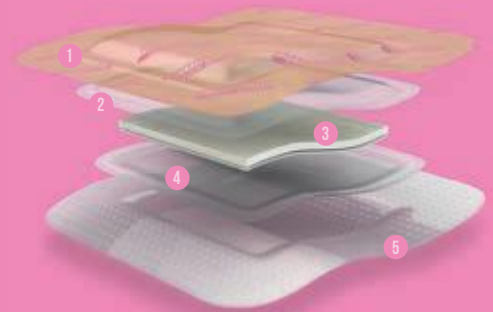

KERRAFOAM GENTLE BORDER			
Product Code	Size (pad size)	Dressings per box	HCPCS
CWL1010	3 x 3 in (2 x 2 in)	10	A6212
CWL1011	4 x 4 in (3 x 3 in)	10	A6212
CWL1013	5 x 5 in (4 x 4 in)	10	A6212
<b>NEW</b> CWL1132	6 x 6 in (5 x 5 in)	10	A6213
<b>NEW</b> CWL1135	7 x 8 in Oval (6 x 7 in)	10	A6213
CWL1012	4 x 8 in (3 x 7 in)	10	A6212
<b>NEW</b> CWL1133	4 x 10 in (3 x 9 in)	10	A6213
<b>NEW</b> CWL1134	4 x 12 in (3 x 11 in)	10	A6213
CWL1023	6.7 x 6.9 in Small Sacrum	5	A6212
CWL1040	9 x 10 in Large Sacrum	5	A6213
CWL1024	9.8 x 9 in Heel	5	A6213

## ALSO AVAILABLE AS KERRAFOAM NON-ADHESIVE

<b>NEW</b> CWL1116	4 x 5 in	10	A6210
<b>NEW</b> CWL1117	6 x 6 in	10	A6210
<b>NEW</b> CWL1118	4 x 8 in	10	A6210
<b>NEW</b> CWL1119	8 x 8 in	10	A6211
<b>NEW</b> CWL1131	8 x 12 in	10	A6211
<b>NEW</b> CWL1120	8 x 17 in	5	A6211

To place an order please contact your medical supply distributor or call 855-522-2211



Crawford Healthcare Inc. | 2005 South Easton Rd. | Suite 203 | Doylestown, PA 18901  
 Tel 855-522-2211 | Email us.info@crawfordhealthcare.com  
[www.crawfordhealthcare.com/us](http://www.crawfordhealthcare.com/us)

KerraFoam™ is a registered trademark of Crawford Woundcare Ltd | © Copyright Crawford Healthcare Ltd, 2016.

1. CHC R160. A fluid retention assessment of KerraFoam Gentle Border and competitor products. Data on File 2014. Crawford Healthcare Ltd. 2. Cooper R. An investigation into the ability of KerraFoam to bind bacteria. Cardiff Metropolitan University. Sept 2013. 3. Evaluation of matrix metalloproteinases by wound care products. Liverpool University. Data on file. 4. CHC-R436. Crawford Ltd. April 2016. 5. CHCR272 Assessment of the dissipation of shear forces by a foam wound dressing. Crawford Ltd. May 2015. 6. Wound Dressing Testing - EN 13726-3 Waterproofness. Crawford Healthcare Ltd. September 2013. 7. Bacterial Barrier Testing. Crawford Healthcare Ltd. August 2013. 8. CHC R186. A product comparison portfolio for KerraFoam Gentle Border. Crawford Healthcare Ltd. May 2015. 9. CHC R421 A healthy volunteer assessment of KerraFoam Gentle Border. Crawford Healthcare Ltd. March 2016. 10. CHC-R250. *In-vitro* assessment of wound dressing bacterial sequestration. Data on File. Crawford Healthcare Ltd. 11. BOLTON, L. 2007. Operational definition of moist wound healing. *Journal of Wound Ostomy and Continence Nursing*, 34, 23-29. Data from Crawford Healthcare Reports CHC-R429. 12. Glover, D., Clark, M. (2015, Sept). Wound management using a superabsorbent foam dressing: outcomes of a post-CE-mark primary care clinical evaluation. *Wounds UK*, 11,3, 77-86. CH14006-REV6