


Suitable for use on low or non-draining wounds, including:

- painful wounds
- leg ulcers
- radiation therapy damage
- burns and scalds
- Epidermolysis Bullosa

Application Guide

- 1 Select a size that allows the dressing to completely cover the wound and extend onto healthy skin (by approximately 1 inch). Non-adhesive Dressing can be cut to shape as needed.
- 2 Prepare the wound site by cleansing and drying the surrounding skin.
- 3 Remove half of the white plastic liner. Position the dressing and smooth into place while removing the remaining liner.
- 4 Secure the dressing with a retention bandage (Non-adhesive only).
- 5 Check the wound regularly. The dressing should be changed as often as the condition of the wound dictates. Note that a change in dressing regimen can result in an initial increased level of exudate which temporarily may require an increased dressing change frequency.

Applying KerraLite Cool (Non-adhesive)

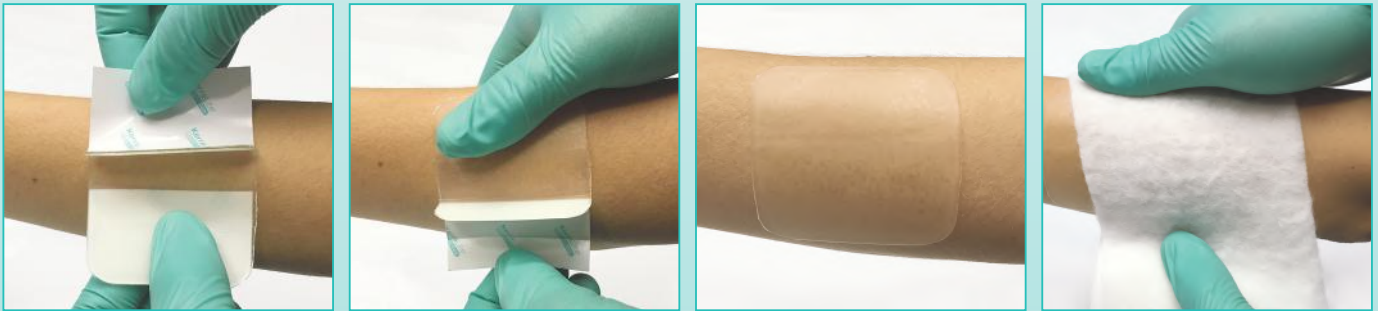

Applying KerraLite Cool (Adhesive Border)





KerraLite[®] Cool dressings are designed to:

- Help to improve healing of lightly exuding wounds
- Clean the wound, autolytically debriding tissue
- Increase granulation tissue of chronic wounds
- Soothe and cool to minimize discomfort
- Protect against wound dehydration
- Barrier to the growth of bacteria
- Help prevent maceration of the peri-wound area
- Can be cut to fit most wounds (non-adhesive only)

Order Information

Product Code	Size	Dressings per box	HCPCS
Non-adhesive			
CWL1004	2.4 x 2.4 in	5	A6234
CWL1005	5 x 3.5 in	5	A6235
CWL1006	7 x 5 in	5	A6235
Adhesive Border			
CWL1007	3 x 3 in	5	A6237
CWL1008	4 x 4 in	5	A6237
CWL1009	6 x 6 in	5	A6238

To place an order please contact your medical supply distributor
or call 855-522-2211



Crawford Healthcare Inc. | 2005 South Easton Rd. | Suite 203 | Doylestown, PA 18901
Tel 855-522-2211 | Email us.info@crawfordhealthcare.com
<http://us.crawfordhealthcare.com>