



### The superpowers of silicone

Silicone is incredibly stable, resists compression extremely well and is odorless, making it the ideal material for KerraPro Pressure Reducing Pads.

### The cost effectiveness of KerraPro

KerraPro pads are comfortable, yet durable, shown to withstand continuous washing - enabling them to be reused on the same patient.

#### Order Information

| Product                | Product Code | Pads per Box |
|------------------------|--------------|--------------|
| Sacrum/Ankle           | KPRO40       | 1            |
| Heel                   | KPRO30       | 2            |
| Sheet 4 x 4 x 0.1 in   | KPRO10       | 5            |
| Sheet 4 x 4 x 0.5 in   | KPRO20       | 5            |
| Strips 12 x 2 x 0.1 in | KPRO50       | 5            |
| Strips 20 x 1 x 0.1 in | KPRO60       | 5            |

To place an order, please contact your medical supply distributor or call 855-522-2211.




Crawford Healthcare Inc. | 2005 South Easton Rd. | Suite 203 | Doylestown, PA 18901  
 Tel 855-522-2211 | Email us.info@crawfordhealthcare.com  
[www.crawfordhealthcare.us](http://www.crawfordhealthcare.us)

KerraPro is a registered trademark of Crawford Woundcare Ltd | © Copyright Crawford Healthcare Ltd, 2016.  
 All other trademarks are property of their respective owners.

# KerraPro<sup>®</sup>


## Pressure Reducing Pads

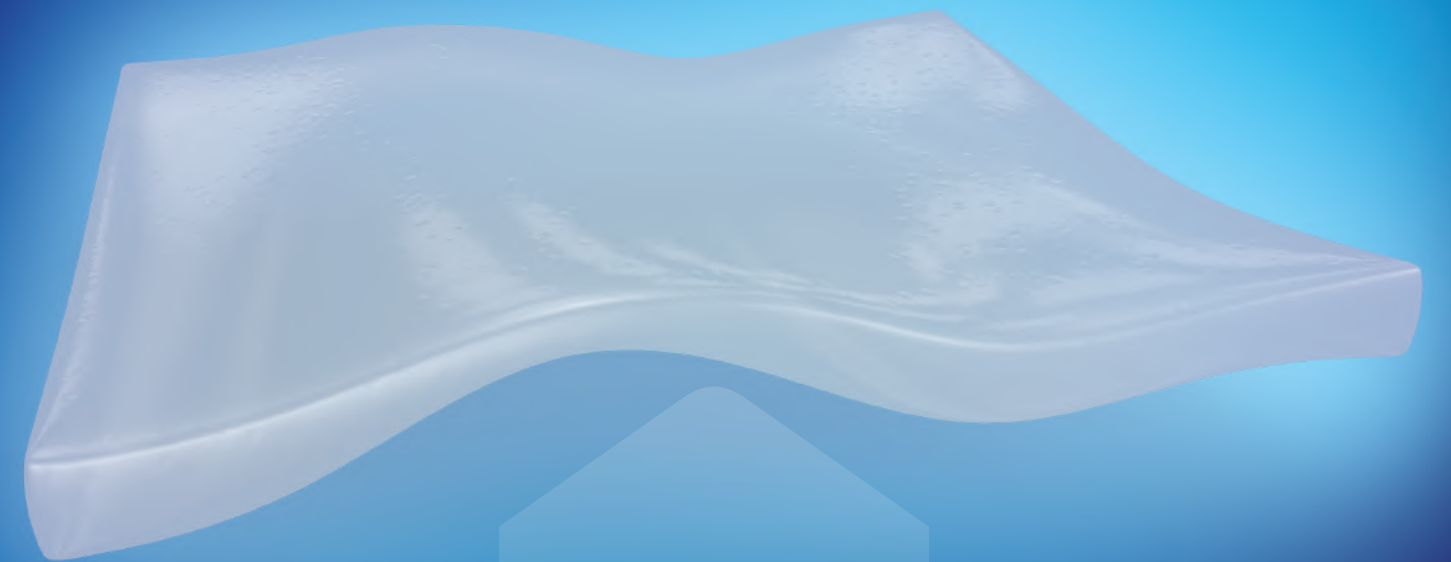




## How KerraPro® protects

KerraPro Pressure Reducing Pads help protect the skin in at-risk patients as part of a pressure ulcer prevention program. Our shaped pads are made from 100% silicone. Silicone is flexible, hardwearing and has the ability to redistribute pressure to protect the skin on bony prominences such as the heel or sacrum.

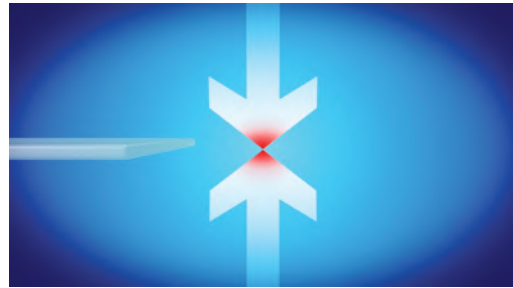

Where the skin rests against the bed or other surface, pressure builds.



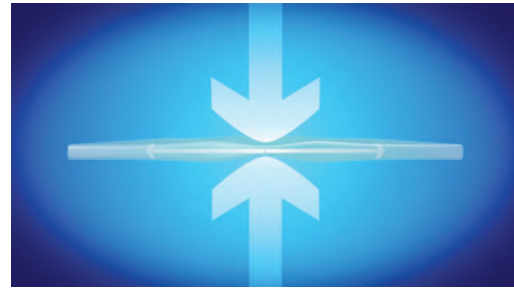


With KerraPro in place, the pressure is absorbed and dissipated throughout the pad.

KerraPro **effectively redistributes this pressure<sup>1</sup>**, dissipating it over the pad to protect the skin from pressure ulcers.



In an *in vitro* lab test, multiple products were tested to observe the pressure reduction using a pressure sensing mat. On one heel, a “pressure reducing” product was used. The other heel (control) had nothing between the heel and the mat.

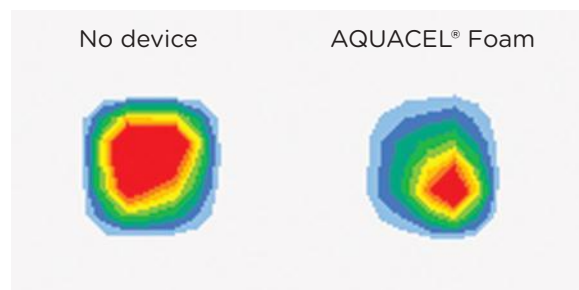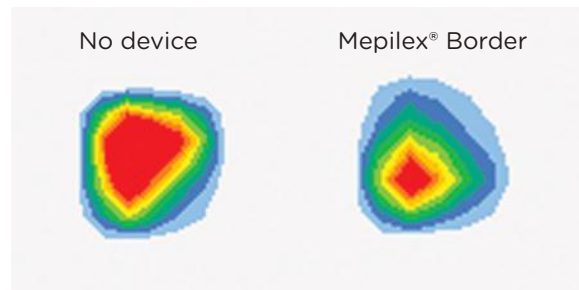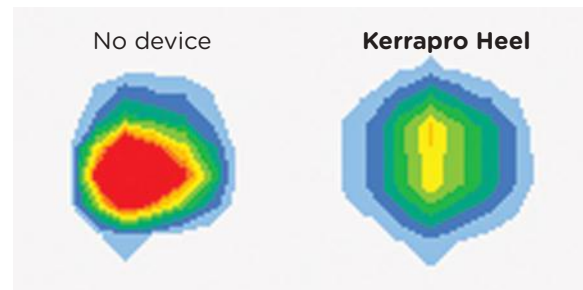

The red in the images indicates higher amounts of pressure while blue indicates a lower amount of pressure.

## The range

KerraPro is available in a range of shapes and thicknesses for a variety of applications.

KerraPro should only be used on intact or recently healed skin to protect it from pressure ulcers.

### 4 x 4 in Sheet 0.1 or 0.5 in thick

KerraPro Sheets can be used flat or folded, and easily cut to the required shape and size for a variety of applications: cushion the head and shoulder blades; reduce pressure between knees and ankles; and protect wrists and hands.




Shoulder Blades



Knees & Ankles



Wrists

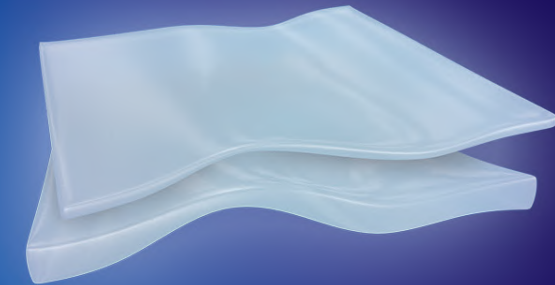

### Heel

KerraPro Heel can be used with the silicone shaped pad directly underneath the heel ① or with the shaped pad at the back of the heel ②. It is also effective in areas such as shoulders, knees and elbows.



Heel ①



Heel ②




Elbow



### Sacrum/Ankle

Perfectly shaped to reduce pressure in the sacral area, KerraPro Sacrum/Ankle provides comfort and protection in particularly bony areas such as the shoulders, hips and ankles.

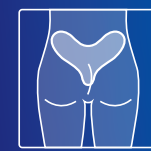


Sacrum



Ankle



Shoulder

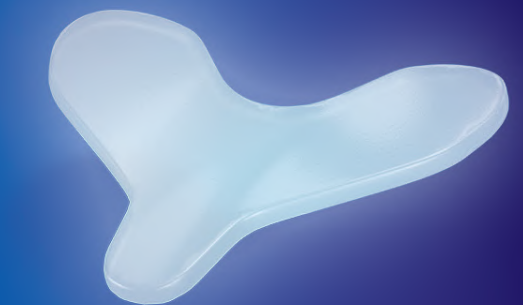

### 0.1in thick Strip 20 x 1 in or 12 x 2 in

KerraPro Strips can be cut to the required length to provide protection when using healthcare equipment such as face masks and tubes. Versatile and flexible, they can also be used as hand, finger, wrist, foot, toe and ankle wraps.



Head



Wrists



Hands

