

The Discreet and Innovative Catheter

Intended Use

Hydrophilic single-use catheter for home or healthcare use
Intermittent catheterization / Bladder emptying

Material Composition

Hydrophilic surface coating: poly(vinyl pyrrolidone)/PVP and table salt/NaCl
Catheter: polyolefin-based elastomer/POBE; Plastic packaging: polyethylene/PE
Water pocket: Laminate foil of poly(ethylene terephthalate)/PET, polyethylene/PE and aluminium, sterile water

Package


Individual "peel-open" packaging
30 catheters per box / 4 boxes per shipping case

Sterilization & Shelf Life

All catheters are sterilized by irradiation. Information about the recommended best-before date is printed on the package.


Directions for Use:

Opening 1 (Using minimal hand dexterity and/or for sterile use)

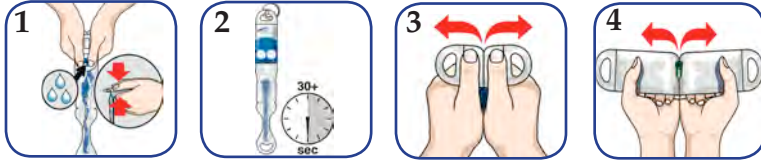

1. Fold the water pouch on the **blue-white border**. Press as shown by applying pressure to both sides.
2. Let the catheter soak for 30 seconds while you prepare for catheterization.
- 3-4. Carefully peel open from the top until you reach the funnel end of the catheter. Remove the catheter from the package and catheterize.

Opening 2 (Using the handling strip for a “no-touch” technique)

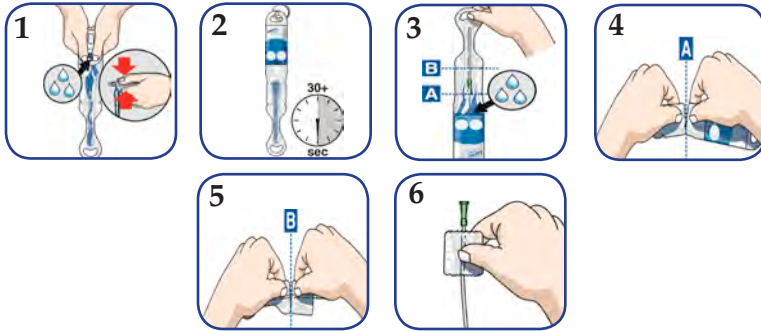

1. Fold the water pouch on the **blue-white border**. Press as shown by applying pressure to both sides.
2. Let the catheter soak for 30 seconds while you prepare for catheterization.
3. Turn the package upside down to return the water into the pouch.
4. Tear at indentation A from both sides to separate the water pouch from the rest of the package.
5. Tear at indentation B from both sides.
6. The remaining part of the package can be used as a handling strip for a hygienic and firm grip during insertion.

Product Range

Item #	Type	Length	Size
4100840	LoFric Primo, Male	16"	8FR
4101040	LoFric Primo, Male	16"	10FR
4101240	LoFric Primo, Male	16"	12FR
4101440	LoFric Primo, Male	16"	14FR
4101640	LoFric Primo, Male	16"	16FR
4101840	LoFric Primo, Male	16"	18FR
9661040	LoFric Primo, Coudé	16"	10FR
9661240	LoFric Primo, Coudé	16"	12FR
9661440	LoFric Primo, Coudé	16"	14FR
9661640	LoFric Primo, Coudé	16"	16FR
9661840	LoFric Primo, Coudé	16"	18FR
4130840	LoFric Primo, Female	8"	8FR
4131040	LoFric Primo, Female	8"	10FR
4131240	LoFric Primo, Female	8"	12FR
4131440	LoFric Primo, Female	8"	14FR
4131640	LoFric Primo, Female	8"	16FR
4131840	LoFric Primo, Female	8"	18FR
4140840	LoFric Primo, Female	6"	8FR
4141040	LoFric Primo, Female	6"	10FR
4141240	LoFric Primo, Female	6"	12FR
4141440	LoFric Primo, Female	6"	14FR
4110640	LoFric Primo, Pediatric	8"	6FR
4110840	LoFric Primo, Pediatric	8"	8FR
4111040	LoFric Primo, Pediatric	8"	10FR

Catheter Funnel-End Color Code

6FR	
8FR	
10FR	
12FR	
14FR	
16FR	
18FR	