


# Melgisorb® Plus

## Absorbent alginate dressing

Melgisorb Plus is a highly absorbent dressing for a wide range of moderately to heavily exuding wounds.

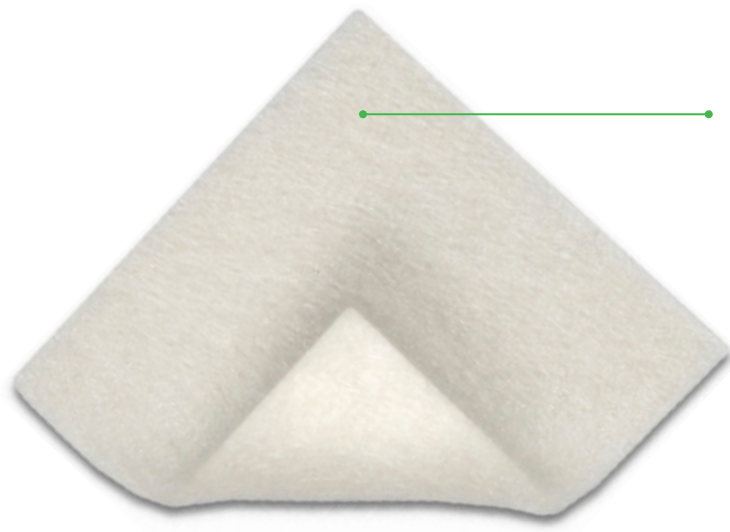


### Hydrophilic alginate fibers


Soft sterile dressing made of Calcium Sodium Alginate fibers

- Excellent absorption - for high exuding wounds
- High wet strength - to allow removal in one piece
- High retention - to limit maceration

## How to use Melgisorb® Plus

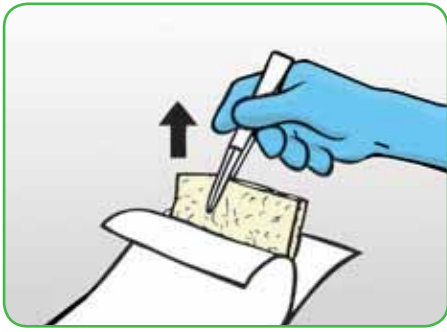


1. Select a size of Melgisorb Plus that is slightly larger than the wound. Open the package by pulling.

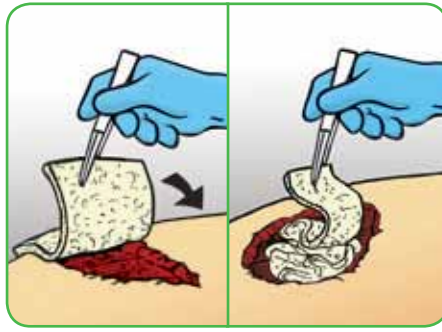

2. Apply a dry Melgisorb Plus product to the moist wound bed.

Shallow wounds: Choose the correct size of flat dressing to be able to cover the entire wound.

Deep wounds and cavities: Cut the appropriate length of the cavity dressing and pack the wound loosely.

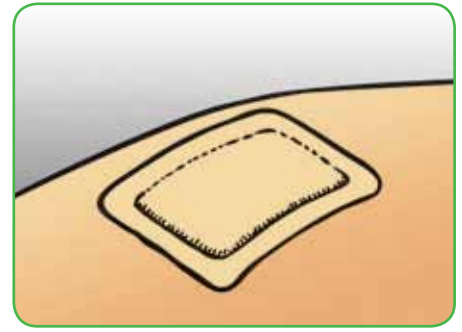

3. Cover and secure Melgisorb Plus with an appropriate secondary dressing such as Meplix® Border.

Compression therapy may be used in conjunction with Melgisorb Plus.

## How Melgisorb Plus works

Melgisorb Plus absorbs large amounts of wound exudate. When absorbing, the alginate fibres form a soft moist gel, giving a moist environment conducive to wound healing. The gel can easily be removed by irrigating with 0.9% saline.

## Benefits of Melgisorb Plus

- Highly absorbent
- Does not adhere to the wounds site when gelled
- Suitable for cavity wounds
- Creates a gel on contact with exudate or solutions containing sodium ions (saline)
- Soft/ pliable/ conformable
- Can be cut to shape
- Non-sensitising

## Precautions

Melgisorb Plus is not indicated for dry wounds, third degree burns or surgical implantations.

If infection is suspected, follow local routines given by health care professionals

If reused performance of the product may deteriorate, cross-contamination may occur.

## Intended Use

Melgisorb Plus is intended for a wide range of moderately to heavily exuding wounds, both infected and noninfected, such as:

- Pressure sores
- Venous and arterial ulcers
- Diabetic ulcers
- Donor sites
- Post operative wounds
- Dermal lesions and other external wounds inflicted by trauma

Configurations available both for flat and cavity wounds

### Melgisorb Plus (Sterile packed)

Art. no	Size cm	Pcs/shelf cont.	PIP-Code	NHS Code
252000	5x5	10	384-2036	ELS585
252200	10x10	10	384-2119	ELS586
252500	10x20	10	384-2101	ELS587
253500	3x45	5	384-2127	ELS588

Note: Not all articles are available in every country. Please contact your local Mölnlycke Health Care representative for information about articles available in your country.



## Find out more at [www.molnlycke.com](http://www.molnlycke.com)

Mölnlycke Health Care AB, Box 13080, SE-40252 Göteborg, Sweden  
Phone +46317223000

The Mölnlycke Health Care, Mepilex® Border names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies.

© 2013 Mölnlycke Health Care AB. All rights reserved.

