


TRIACT Foam With Silicone Border.

The peri-wound deserves silicone.
The wound deserves better.

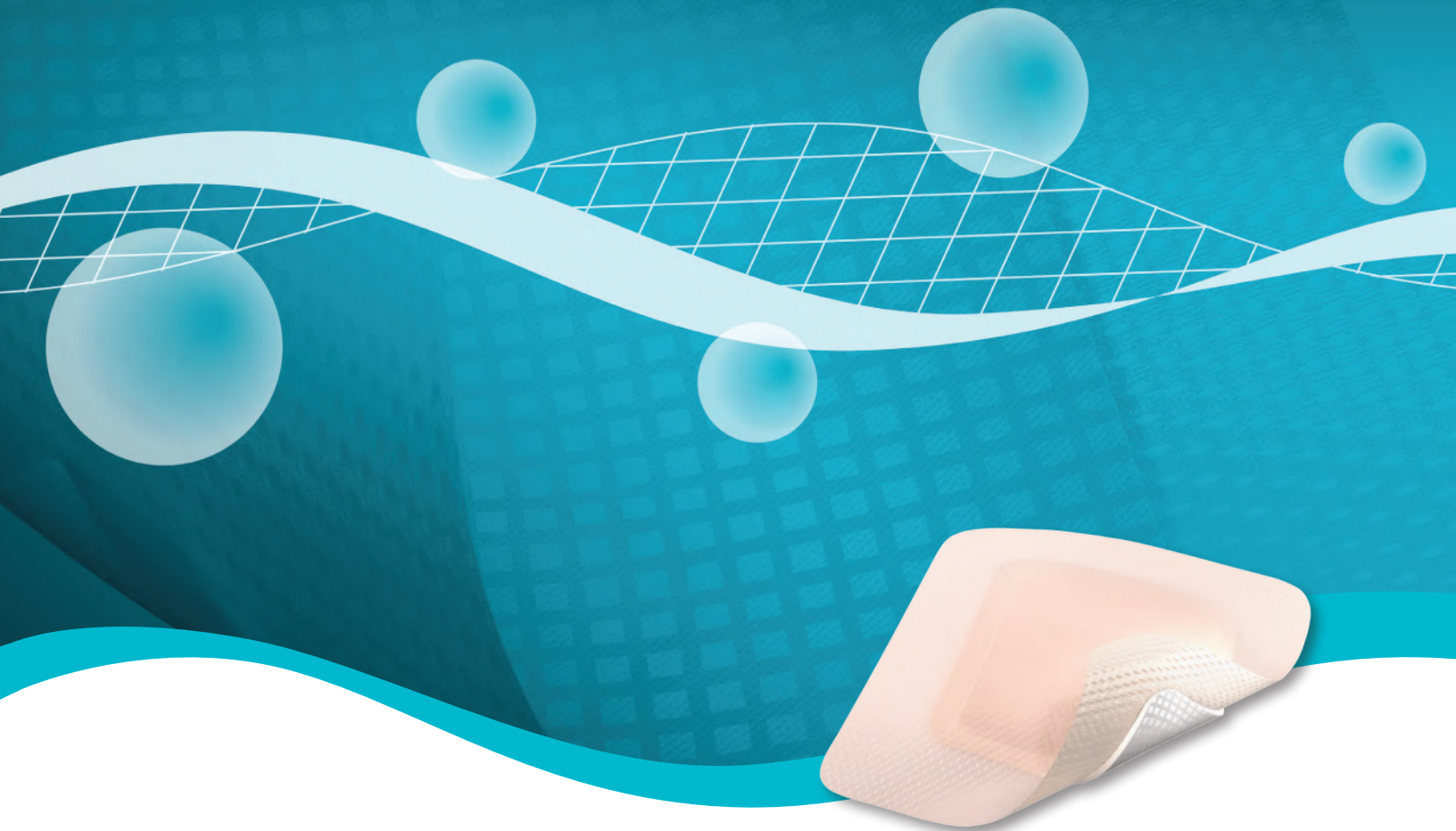


- TRIACT Technology helps create a moist, wound-healing interface
 - TRIACT Technology helps prevent damage to newly formed tissue during dressing removal
 - Silicone border offers virtually pain-free removal



The TRIACT Difference

Lipido-colloid matrix contains CMC particles that form a gel in the presence of exudate, which:

- Helps provide a moist, wound-healing environment
- Helps provide a non-adherent wound interface
- Helps prevent damage to newly formed tissue during dressing removal

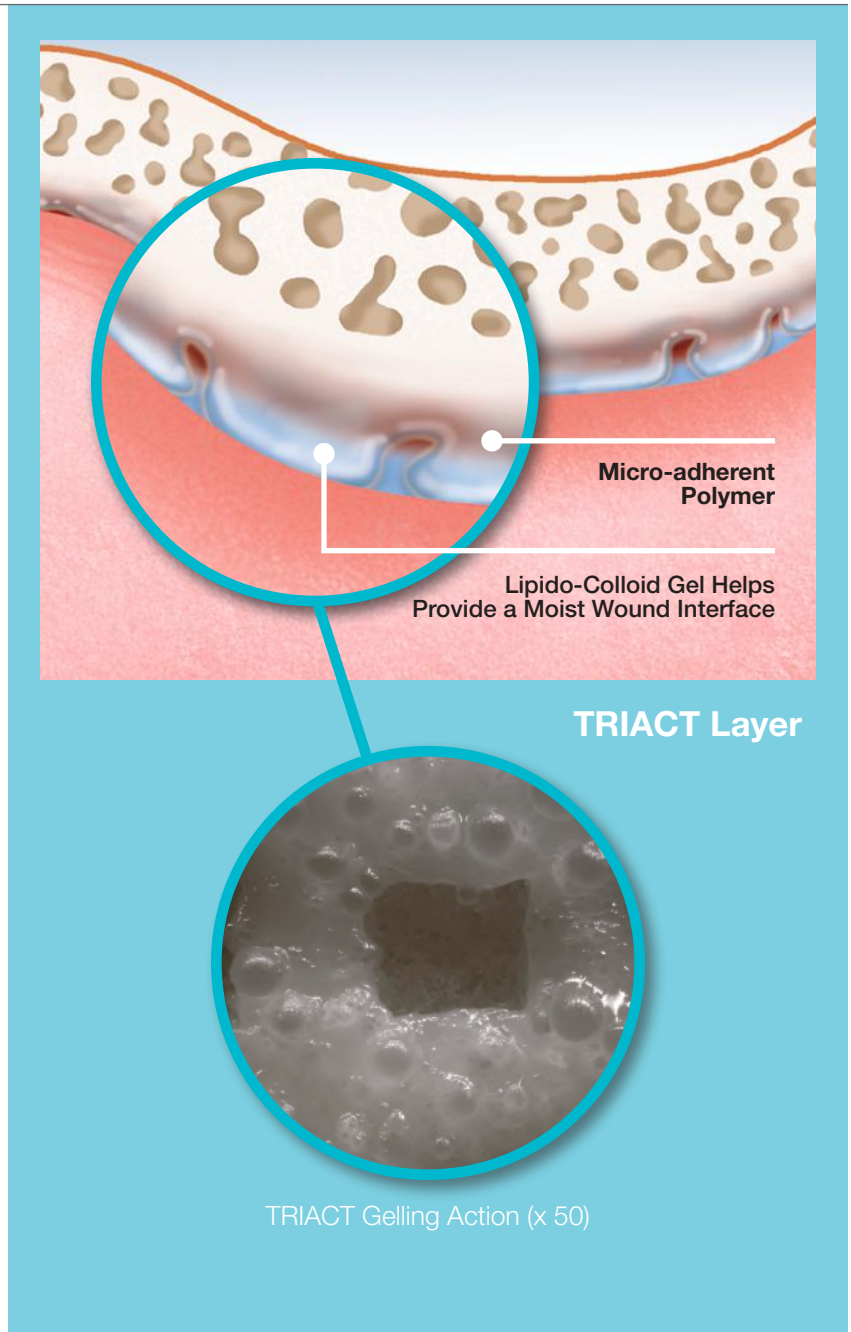

After 72 h, no gelling action is observed in Allevyn Gentle Border or Mepilex Border.¹

Allevyn Gentle Border 2-D Observation (x 50)


Silicone after contact with fluids for 72 h



Silicone Layer Magnified

Mepilex Border 2-D Observation (x 50)


Silicone after contact with fluids for 72 h

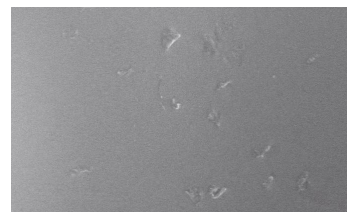



Silicone Layer Magnified

Silicone adhesive protects the peri-wound.

TRIACT layer provides a moist wound environment.

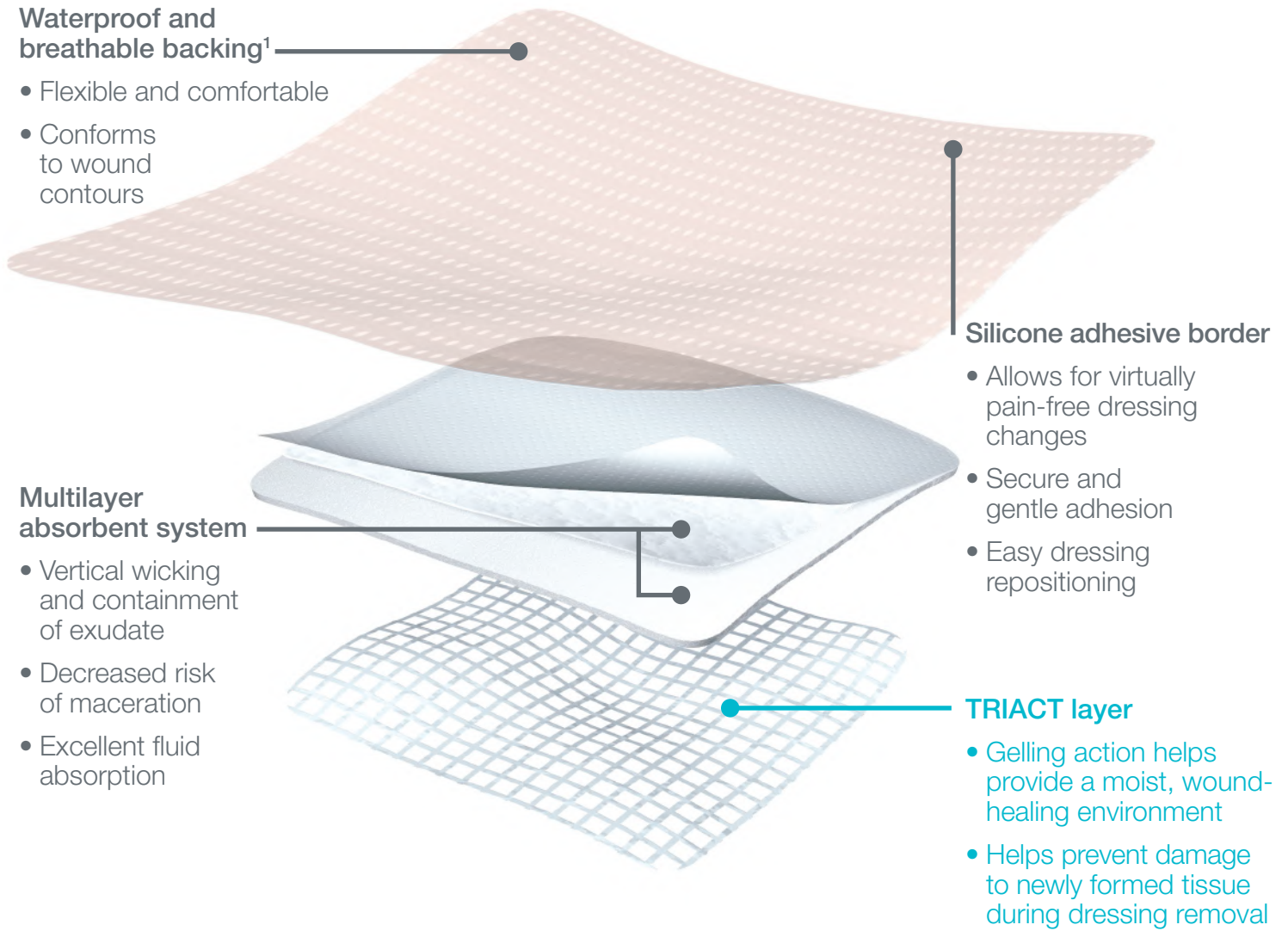

TRIACT foam with silicone border demonstrates a high absorption capacity.¹

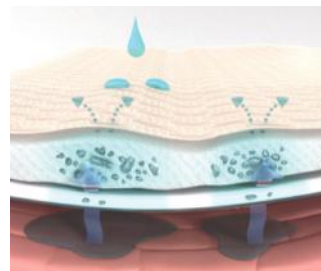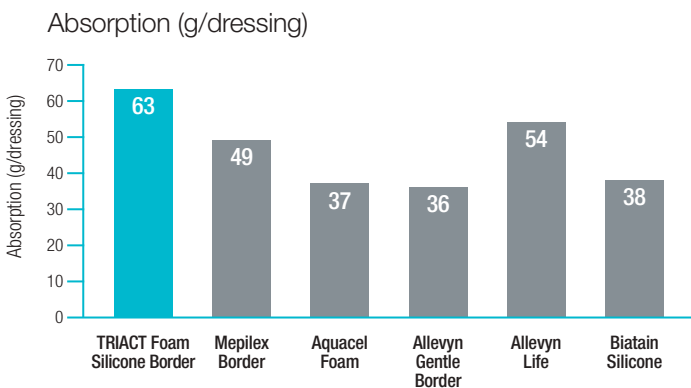

Dressings featuring TRIACT Technology

TRIACT Foam With Silicone Border

Stock No	Product Size	Quantity/Box	HCPCS
550762	4" x 4" (10 cm x 10 cm) Pad 2.5" x 2.5" (6 cm x 6 cm)	10	A6212
550763	6" x 6" (15 cm x 15 cm) Pad 4" x 4" (10 cm x 10 cm)	10	A6212
550764	6" x 8" (15 cm x 20 cm) Pad 4" x 6" (10 cm x 15 cm)	10	A6213
550765	8" x 8" (20 cm x 20 cm) Sacral Pad 5" x 5.5" (12.5 cm x 14.5 cm)	10	A6213



Restore Foam With TRIACT Technology Non-Border

Stock No	Product Size	Quantity/Box	HCPCS
509380	2.5" x 2.5" (6 cm x 6 cm)	10	A6209
509381	4" x 4" (10 cm x 10 cm)	10	A6209
509382	6" x 6" (15 cm x 15 cm)	10	A6210
509384	4.7" x 7.5" (12 cm x 19 cm) Heel	10	A6210



Restore LITE Foam With TRIACT Technology Non-Border

Stock No	Product Size	Quantity/Box	HCPCS
509393	2.5" x 2.5" (6 cm x 6 cm)	10	A6209
509394	4" x 5" (10 cm x 12.7 cm)	10	A6210
509395	6" x 6" (15 cm x 15 cm)	10	A6210
509396	6" x 8" (15 cm x 20 cm)	10	A6210



*Restore Silver Foam With TRIACT Technology Non-Border

Stock No	Product Size	Quantity/Box	HCPCS
509345	4" x 4" (10 cm x 10 cm)	10	A6209
509346	6" x 8" (15 cm x 20 cm)	10	A6210



Restore Contact Layer FLEX With TRIACT Technology

Stock No	Product Size	Quantity/Box	HCPCS
506487	2" x 2" (5 cm x 5 cm)	10	A6206
506488	4" x 5" (10 cm x 12.7 cm)	10	A6207
506489	6" x 8" (15 cm x 20 cm)	10	A6207



Restore Contact Layer Silver With TRIACT Technology

Stock No	Product Size	Quantity/Box	HCPCS
509341	4" x 5" (10 cm x 12.7 cm)	10	A6207
509342	6" x 8" (15 cm x 20 cm)	10	A6207



1. Data on file 2014.

Manufactured for:
Hollister Incorporated
 2000 Hollister Drive
 Libertyville, IL 60048 USA
www.hollisterwoundcare.com

*Caution: For dressings which contain silver, Federal (USA) law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

For product questions, sampling needs, or detailed clinical questions concerning our products in the US call **1.888.740.8999**.

The Hollister logo, the TRIACT logo, and "The wound deserves better." are trademarks of Hollister Incorporated.

All other trademarks are the property of their respective owners.

Made in France.

©2015 Hollister Incorporated.

922510-0815

