

3M™ Tegaderm™ Superabsorber Dressing

Tegaderm™ Superabsorber Dressing



Locks in Exudate

Tegaderm™ Superabsorber Dressing provides confidence that the dressing will absorb and contain exudate — even under compression.

- Versatile for a variety of wounds with moderate and high exudate levels
- Helps prevent maceration of the wound edges
- Reduces frequency of dressing changes
- Provides patient comfort

3M

Innovative Design Absorbs and Contains Exudate — Even Under Compression

Manage Highly Draining Wounds with Confidence



**Diabetic
(Neuropathic)
Foot Ulcer**



**Venous
Leg Ulcer**



**Dehiscd
Surgical
Wound**



**Pressure Ulcer
Stage III**



**Pressure Ulcer
Stage IV**



**Pressure Ulcer
Unstageable due
to necrotic base**

What does Tegaderm™ Superabsorber Dressing do?


- Absorbs and retains exudate in the dressing minimizing the risk of maceration of the wound edges¹, even under compression.
- Maintains a moist wound environment.
- Removes exudate away from the skin surface through rapid, vertical wicking.
- Traps the wound fluid and wound fluid components away from the wound².



before it becomes saturated.


1. Clinical experiences of using a super absorbing dressing. Meuleneire F. Poster. EWMA Congress 2010, Geneva, Switzerland.

2. The inhibition of matrix metalloproteinase activity in chronic wounds by a polyacrylate superabsorber. Eming S, Smola H, Hartmann B, Malchau G, Wegner R, Krieg T, Smola-Hess S (2008) Biomaterials Jul 29 (19): 2932-2940

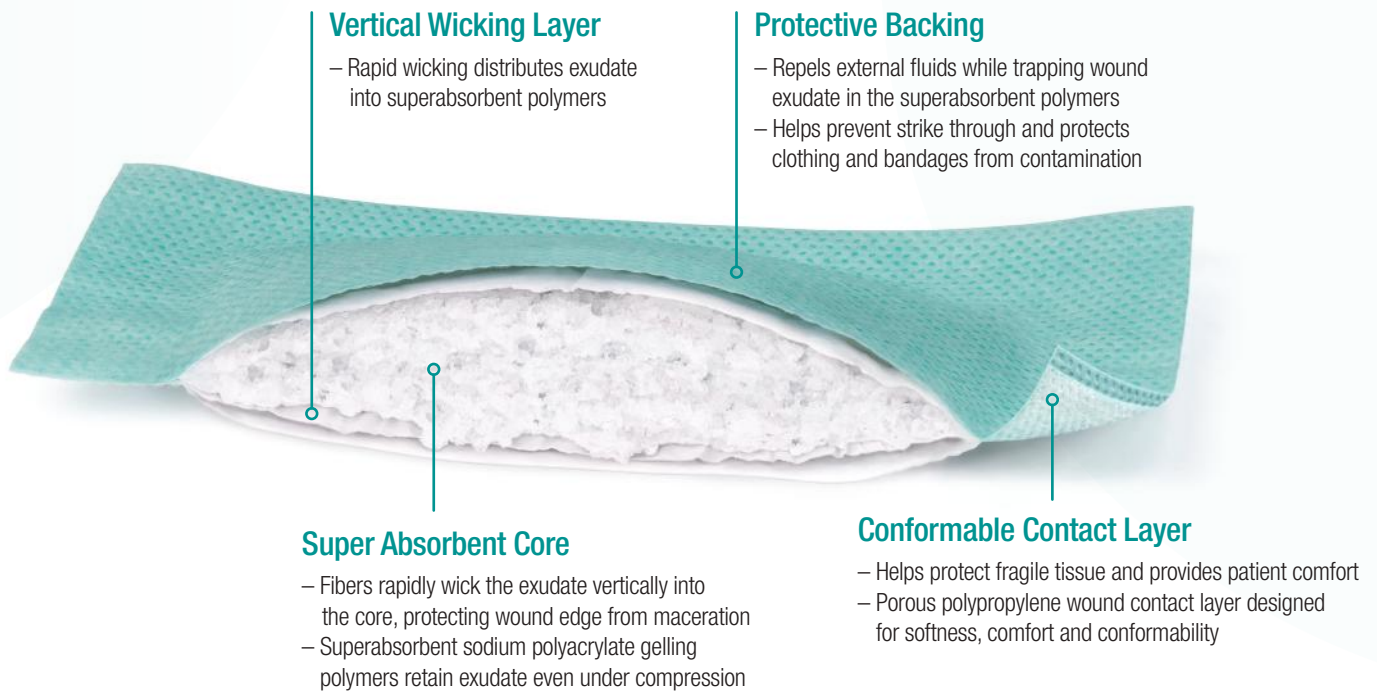


3M™ Tegaderm™ Superabsorber Dressing Helps Prevent Maceration

Mean (SEM) Free Swell Absorbency [g/100 cm²]

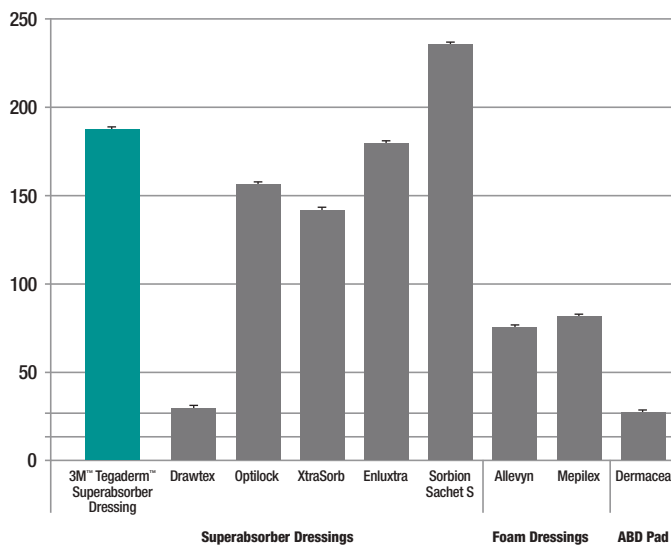

Data on file at 3M.

Mean (SEM) Fluid Retention Capacity [g/g]

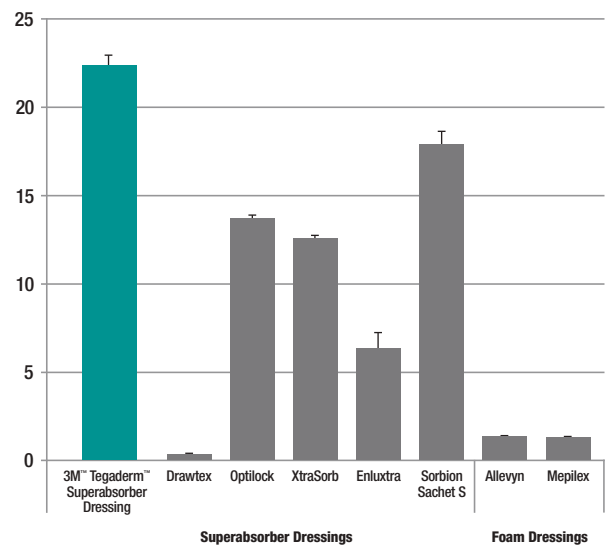

Data on file at 3M.


High absorption and superior exudate retention provides confidence to manage exudate and supports patient comfort with fewer disruptions. The innovative design of Tegaderm™ Superabsorber Dressing helps prevent complications related to periwound maceration.

Easy and Efficient to Use

3M™ Tegaderm™ Superabsorber Dressing

Tegaderm™ Superabsorber dressing can be used both as a primary or secondary dressing. Deep wounds require a wound filler to be applied, since the Superabsorber dressing may not be used as a sponge.

- 1 Select a dressing size that allows 1 inch (2.5 cm) overlap onto intact skin.
- 2 For wounds with depth, use absorbent wound fillers such as 3M™ Tegaderm™ Alginate Dressing or 3M™ Tegaderm™ Alginate Ag Silver Dressing.
- 3 Apply dressing with white contact layer towards the wound — the green hydrophobic backing protects against strike through.
- 4 Secure the dressing by taping the edges with preferred adhesive or other securement, e.g. 3M™ Kind Removal Silicone Tape, 3M™ Tegaderm™ Transparent Film Roll, 3M™ Medipore™ H Soft Cloth Surgical Tape or under 3M™ Coban™ 2 Layer Compression System.

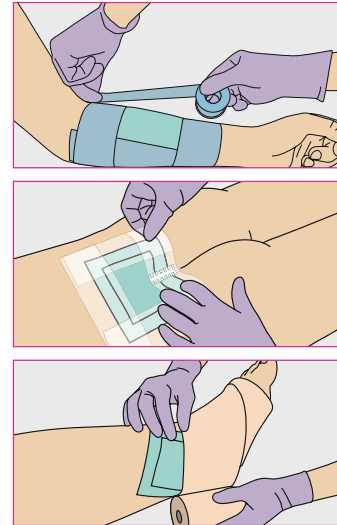

3M™ Tegaderm™ Superabsorber Dressing is compatible with other 3M products

- 3M™ Cavilon™ No Sting Barrier Film
- 3M™ Coban™ 2 Layer Compression System
- 3M™ Kind Removal Silicone Tape
- 3M™ Medipore™ H Soft Cloth Surgical Tape
- 3M™ Tegaderm™ Ag Mesh Dressing with Silver
- 3M™ Tegaderm™ Alginate Ag Silver Dressing
- 3M™ Tegaderm™ Alginate Dressing
- 3M™ Tegaderm™ Transparent Film Roll
- 3M™ Wound Cleanser

Ordering Information

| Item Number | Overall Dressing Size | Superabsorber Size | Items/Box | Boxes/Case | HCPCS Code* |
|-------------|--|--|-----------|------------|-------------|
| 90701 | 3-7/8 in. x 3-7/8 in. (10 cm x 10 cm) | 2-3/4 in. x 3-1/8 in. (7 cm x 8 cm) | 10 | 5 | A6196 |
| 90702 | 3-7/8 in. x 7-7/8 in. (10 cm x 20 cm) | 3-1/8 in. x 6-5/8 in. (8 cm x 17 cm) | 10 | 5 | A6197 |
| 90703 | 7-7/8 in. x 7-7/8 in. (20 cm x 20 cm) | 6-1/4 in. x 6-5/8 in. (16 cm x 17 cm) | 10 | 5 | A6197 |
| 90704 | 7-7/8 in. x 11-3/4 in. (20 cm x 30 cm) | 6-1/4 in. x 10-5/8 in. (16 cm x 27 cm) | 10 | 5 | A6198 |

* Disclaimer: The reimbursement information provided by 3M Health Care and its representatives is intended to provide general information relevant to coding for 3M products. Insurers' reimbursement policies can vary, however, and the use of the codes discussed here does not guarantee that an insurer will cover or pay at any particular level.

For more information, contact your 3M Health Care representative, call the 3M Health Care Customer Helpline at 1-800-228-3957 or visit us at www.3M.com/C3SD.



Critical & Chronic Care Solutions Division
3M Health Care
 2510 Conway Avenue
 St. Paul, MN 55144
 USA
 1-800-228-3957
www.3M.com/C3SD

3M™ Tegaderm™ SIMPLE. DEPENDABLE. TRUSTED.
 Dressings

3M, Cavilon, Coban, Medipore and Tegaderm are trademarks of 3M. Allewyn is a trademark of Smith & Nephew, Dermacea is a trademark of Covidien, Drawtex is a trademark of Beier Drawtex Healthcare, Enluxtra is a registered trademark of BASF, Mextra and Mepilex are trademarks of Mölnlycke, Optilock is a trademark of Medline, Xtrasorb is a trademark of Derma Sciences. Please recycle. Printed in U.S.A.
 © 3M 2013, 2014. All rights reserved. 70-2010-9302-1